




# ULTRASONIC INSERT POWER LEVEL GUIDE

Get the Right Power from Your Inserts

*Sometimes highest power isn't the best power.*  
Use your inserts within the recommended power ranges for maximum power scaling efficiency and effectiveness and to extend the inserts' useful life.

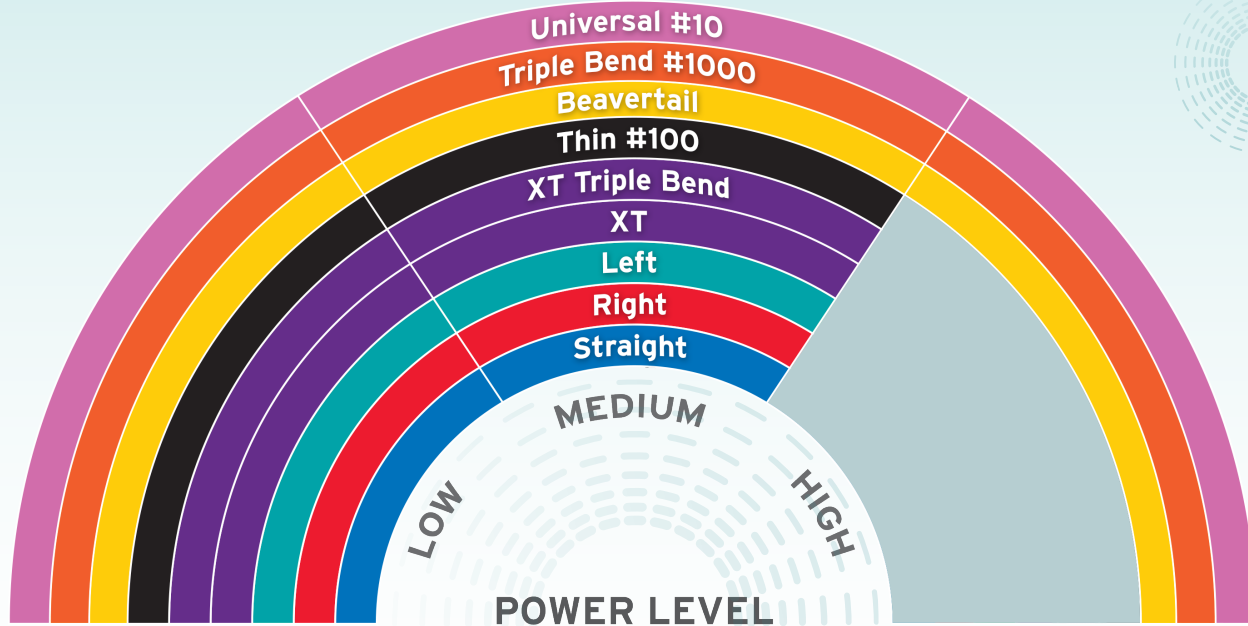

How the best perform

**Hu-Friedy**

# SWERV<sup>3</sup> ULTRASONIC SCALER POWER LEVEL GUIDE

To ensure patient comfort and to extend the life of your inserts, always use the lowest effective power setting.



## Green Power Mode

Recommended Power Settings: 3-10  
Light to moderate deposits



## Blue Power Mode

Recommended Power Settings: 1-10  
Moderate to heavy deposits

Learn more at [Hu-Friedy.com/Inserts](https://www.hu-friedy.com/inserts)

To reorder inserts call **1-800-Hu-Friedy** or your authorized Hu-Friedy dealer.