

# ConFirm™ Rapid Read 20-Minute System



Infection Prevention and  
Instrument Management

The ConFirm™ Rapid Read 20-Minute System determines the efficacy of steam sterilization cycles in just 20 minutes. An enzyme-based detection system between the biological indicator and reader is used to provide the results.

**When frequent sterilizer testing is needed and time is of the essence, release an instrument load into service in just 20 minutes.** Faster biological test results help mitigate risk and potential recalls due to improperly functioning sterilizers.



Twist & Flick Activation reduces contamination potential and **eliminates need for a separate activator device**



Vials contain *Geobacillus stearothermophilus* bacterial spores to facilitate **detection of unsafe outcomes**

Media sealed in cap with foil allows for **ease of activation**


BI information (lot number, expiration date) for **easy documentation**

Type 1 chemical indicator **distinguishes between processed and unprocessed BIs**



Glass-free design **helps ensure staff safety**

Purple media gives clear visual indication that media is in the vial after activation and **reduces chance of error**

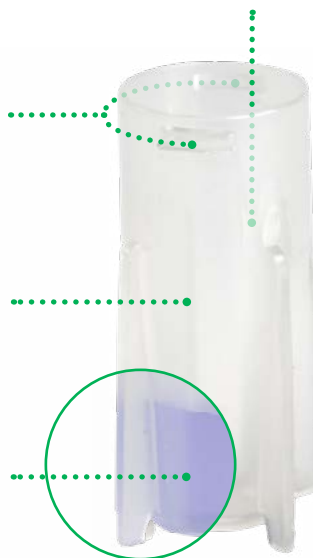


8-well design provides **ample space for testing multiple sterilizers**

LCD screen **displays remaining incubation time and lists incubation results**

Barcode scanner **reads product number, lot #, and expiration date**





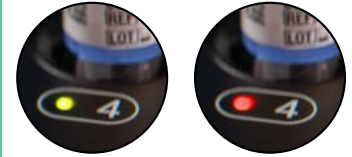

**Results are read automatically** and indication given via lights at each well



**Pass**

**Fail**

#### ADDITIONAL FEATURES

-  **Set language**
-  **Volume adjustment**
-  **Connectivity ports** for network and printer connections
-  **Power loss back-up** allows 60-second recovery time if device loses power



x 25

### ConFirm™ Rapid Read 20-Minute Reader

SKU: **RR20READER**

- Compact, 8-well, tabletop incubator incubates and automatically reads ConFirm™ Rapid Read Steam BIs at 57°C (135° F)
- Fluorescent results within 20 minutes

### ConFirm™ Rapid Read 20-Minute Biological Indicators, 25 Count

SKU: **RR20BI**

- Self-contained biological indicators
- Determines steam sterilization efficacy of commonly used cycles
- Results within 20 minutes with the ConFirm™ Rapid Read BI Reader

Learn More at [HuFriedyGroup.com](https://www.HuFriedyGroup.com)

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.  
2024 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-908/2405