

AIR-FLOW[®] HANDY 3.0 UNITS



BETWEEN PATIENTS

- Remove and clean Handy handpiece with Easy-Clean
- For Handy 3.0 Perio handpiece, remove nozzle first
- Soak handpiece for ten minutes in enzymatic solution
- Wipe Handy body with a surface disinfectant
- Rinse handpiece with purified water
- Let handpiece dry
- Autoclave Handy handpiece (recommended sterilization is 132°C/270°F for 4 minutes and a drying minimum of 20 minutes)
- After sterilization, moisture may be present in the handpiece. Blow the interior of the handpiece with compressed air to avoid the formation of powder concretions in the air tubes

BEFORE PATIENTS

- Replace disposable perio nozzle tip (Handy 3.0 Perio handpiece only)
- Refill Powder chamber

END OF THE DAY

- Repeat cleaning steps listed under "Between Patients"
- Empty powder chamber
- Use evacuation or air to remove residual powder from chamber, cap and threading

LONG TERM MAINTENANCE

- Replace O-Rings of the chamber cap when damaged or worn.
- Replace o-rings on nozzle tube when damaged or worn

DON'TS

- ❗ Do not use an abrasive sponge or scouring powder to clean
- ❗ Do not submerge Handy body in liquid or autoclave
- ❗ Do not let any liquid into powder chamber
- ❗ Do not leave powder in the powder chamber at the end of the day
- ❗ Do not reuse PERIO-FLOW nozzle tips – these are single use only.

