




HIGH-LEVEL DISINFECTANT

# SANI Glut™

Glutaraldehyde 3%

## When safety counts and minutes matter.

**SANI Glut™** Glutaraldehyde (3%) with activator is a reusable sterilizing and high-level disinfection solution. **SANI Glut** is non-corrosive and is compatible with immersible, heat sensitive instruments and devices. Once activated, **SANI Glut** can be used up to 28 days\* **SANI Glut** provides high-level disinfection in 25 minutes and sterilization in 10 hours

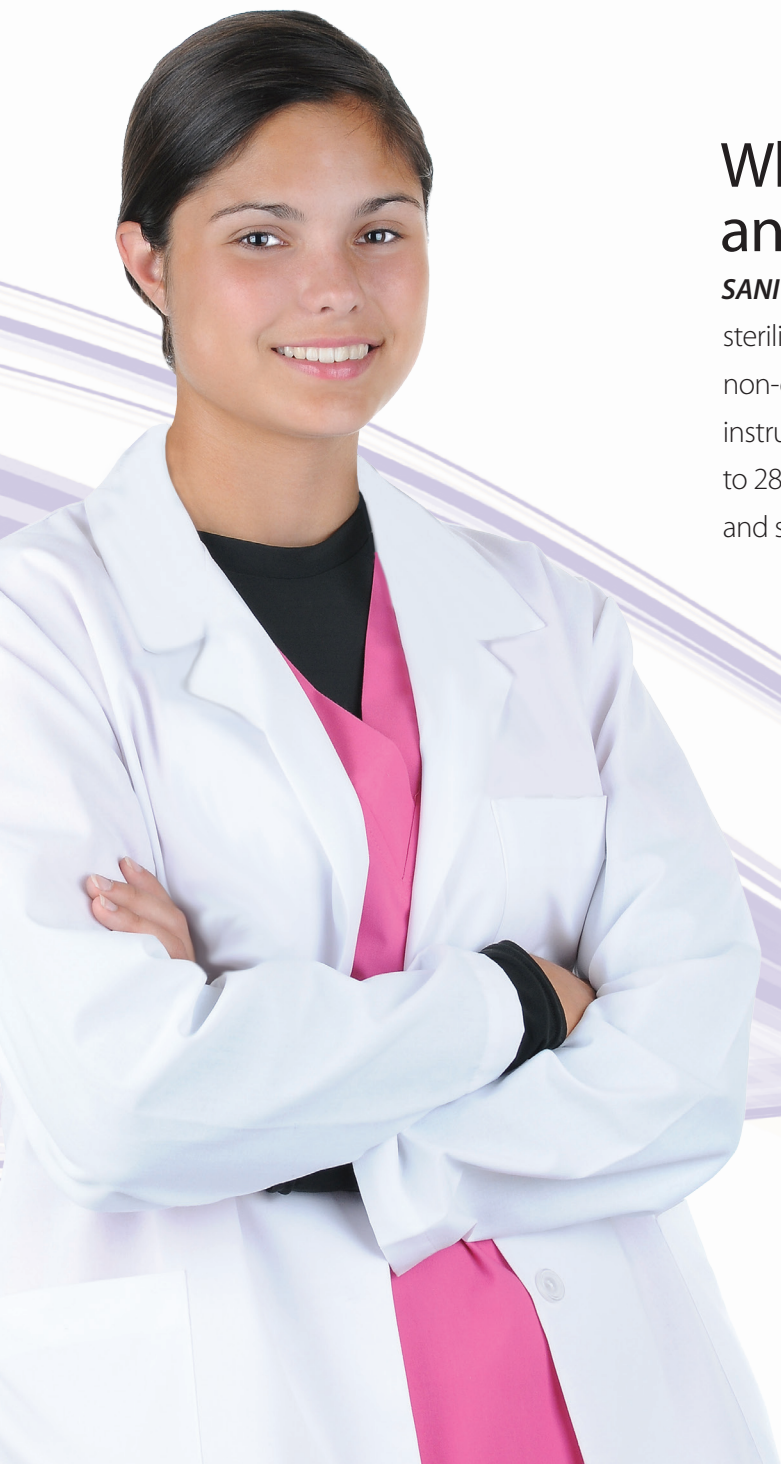

**SANI Glut™**  
Glutaraldehyde 3%

28 Day Reuse  
25 Minute High Level Disinfection  
10 Hour Sterilization

Reusable Sterilizing and High Level Disinfecting Solution

Glutaraldehyde	3.0%
Other Ingredients	97.0%
Total	100.0%

Keep Out Of Reach Of Children  
DANGER: See side panel for additional precautionary statements

Contents: 1 Gallon (128 FL. OZ./3.8 L)

**CROSSTEX** SPS  
A CANTEL MEDICAL COMPANY medical

# The *SANI Glut*<sup>TM</sup> Advantage

## FAST ACTING AND EFFECTIVE

*SANI Glut*<sup>TM</sup> Glutaraldehyde is fast acting and useful against a variety of microorganisms. It is readily biodegradable, formaldehyde-free and performs more effectively than products that use formaldehyde, glyoxal or other corrosive materials.

## FEATURES & BENEFITS

- **EFFECTIVE** – achieves high-level disinfection in 25 minutes and sterilization in 10 hours.
- **CONVENIENT** – once activated, ready to use.
- **ECONOMICAL** – reusable for up to 28 days\*.
- **EXCELLENT MATERIALS COMPATIBILITY** – non-corrosive and compatible for use on heat sensitive instruments and devices.
- **EASY ACTIVATION** – activate *SANI Glut* by adding the entire contents of the activator vial which is attached to the *SANI Glut* container.

## TECHNICAL INFORMATION

- In-Use Concentration: 3% Glutaraldehyde
- High-Level Disinfection: 25 minutes at 25°C
- Sterilization: 10 hours at 25°C
- Use Life: Up to 28 days\* once activated
- Use Conditions: Designed for manual use
- Shelf Life: 2 years
- Storage: 15-25°C
- Disposal: Drain as per facility policy. Flush thoroughly with water



\* Solution needs to be tested with the appropriate test strip prior to each use to verify effective disinfection concentration.

## BY THE NUMBERS

**28** DAY reuse period\*

**25** MINUTE minimum high-level disinfection time at 25°C

**10** HOUR sterilization at 25°C



## Ordering Information

Description	Quantity	Ref.#
<i>SANI Glut</i> <sup>TM</sup> Glutaraldehyde 3% with activator	1-gallon bottle	JGLUT
SANI HLD Holder	Holds 1 gallon of liquid	JHOLD



Manufactured for Crosstex  
10 Ranick Road, Hauppauge, NY 11788-4209 USA  
Tel: 631-582-6777 | 888-276-7783 | Fax: 631-582-1726  
E-mail: [crosstex@crosstex.com](mailto:crosstex@crosstex.com) | [crosstex.com](http://crosstex.com)

California • Colorado • Georgia • New York • Pennsylvania • Argentina • Japan • Singapore • The Netherlands