

# Sani ProZyme™ Enzymatic Detergent

REF JED

EN

## INSTRUCTIONS FOR USE

### INSTRUCTIONS FOR USE:

Wear gloves and eye protection.

**Presoak & Manual Washing:** For light organic debris, dilute 0.5 fl oz (15 mL) of concentrated solution per gallon (4 L) of warm water. For normal organic debris, dilute 1 fl oz (30 mL) of concentrated solution per gallon (4 L) of warm water. For heavy organic debris, dilute 2 fl oz (60 mL) of concentrated solution per gallon (4 L) of warm water. Soak instruments for 2 to 3 minutes or longer, if required. Soak 15 to 20 minutes for heavy organic debris. Solution should be aspirated through the lumens/channels of instruments. Scrub instruments and rinse thoroughly with warm water, aspirating water through lumens/channels to completely remove all traces of enzymatic solution. If equipment is to be chemically disinfected, dry before immersion into the solution. Discard solution after each use.

**Ultrasonic & Automated Washers:** Determine the appropriate ratio of detergent-to-water based upon instructions for presoak and washing above and follow equipment manufacturer's instructions. Recommended temperature range: 90 °F to 140 °F (32 °C to 60 °C). It is recommended to use validated cleaning procedures that are in compliance with best practices.

### Keep Out Of Reach Of Children.

See SDS for additional information.



### DANGER.

May cause allergy or asthma symptoms or breathing difficulties if inhaled. Precautionary Statement(s): Avoid breathing dust/fume/gas/mist/vapors/spray. In case of inadequate ventilation wear respiratory protection. **If inhaled:** If breathing is difficult, remove person to fresh air and keep comfortable for breathing. **If experiencing respiratory symptoms:** Call a poison center or doctor. Dispose of contents and container in accordance with all local, regional, national and international regulations. 4% of the mixture consists of ingredient(s) of unknown acute toxicity.

### FOR EMERGENCY MEDICAL & SAFETY INFORMATION

CALL (24 hrs) 1-800-535-5053.


INTERNATIONAL: 1-352-323-3500



Manufactured for: Crosstex International, Inc.  
6789 W. Henrietta Road, Rush, NY 14543 USA  
585.359.0130 | 631.582.6777 | crosstex.com

Made in USA  
JED-IFU-EN / REV C - 09/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.  
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

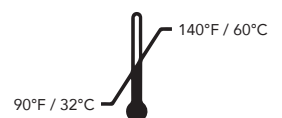

**CROSSTEX**