# PWR Perio Powder

EN

**REF** AP-PM / AP-PN

## INSTRUCTIONS FOR USE

### PRODUCT DESCRIPTION

PWR Perio Prophylaxis Powder for supragingival and subgingival treatments.

Before using this product, please read the unit-specific operating instructions carefully and follow the recommendations contained therein. Please pay special attention to the safety precautions.

### INDICATION FOR USE

Prophylaxis powder based on non-cariogenic glycine (approx. 25 $\mu$ ). For the gentle removal of supragingival and subgingival plaque and discoloration on teeth by using a dental powder blasting unit, which is suitable for the application of abrasives with an average grain size between 10 $\mu$  and 80 $\mu$ .

### OPERATION METHOD

1. Before opening the cap, shake the bottle up and down with hands for 3-5 times.
2. Open the cap and pour the appropriate amount of powder into the powder tank of the device according to the patient's oral condition, and then tighten the cap.
3. Activate the device and align the nozzle with patient's tooth surface to start scaling.

### CONTRA-INDICATIONS

- Patients and users suffering from chronic bronchitis or asthma must not, under any circumstances, be treated or use an air polishing device. The jet of air and powder could cause respiratory difficulties.
- Do not use if the patient is glycine-sensitive.
- Patients who are suspected of being allergic to flavorings should not be treated with the powder.
- Note that powder is provided non-sterile. Please take this information into consideration when performing procedures. The powder should only be used for non-surgical procedures.
- The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients:
  - Patients suffering from upper respiratory tract infections.
  - Pregnancy, breast feeding.
  - Patients under treatment (radiotherapy, chemotherapy, antibiotics).
  - Immune deficiency (neutropenia, agranulocytosis, diabetes, hemophilia).
- Mint flavor only: The flavors contained in the powder can lead, in specific cases, to allergic reactions. Patients who are suspected of being allergic to flavorings must not be treated with the powder.

### RECOMMENDATIONS FOR THE TREATMENT

- Shake well before use.

### WARNING AND PRECAUTIONS

- Do not swallow.
- Avoid contact with the eyes. In case of contact with the eyes, rinse with plenty of water.
- Patients and operators must wear safety goggles.
- The operator must wear a mask and gloves.
- Use a high-performance suction device.
- Apply Vaseline® on the patient's lips.
- Wash hands after use.

### GENERAL INFORMATION

- Do not exceed the powder filling limit as indicated on the powder chamber.
- This powder is sugar free.
- If the bottle is left opened, there is a risk of agglomeration of the powder, that could block the device.

**STORAGE AND SHELF LIFE**

- The powder should not be stored near heat sources.
- Storage condition: 14°F to 86°F (-10°C to 30°C) and 10% to 70% relative humidity.
- Transport conditions: 14°F to 86°F (-10°C to 30°C) and 10% to 90% relative humidity.
- Do not use after expiration date.
- The powder should not remain more than 3 days inside the closed powder chamber.

**INGREDIENTS:**

Glycine, Amorphous silica.  
Mint flavor only: Aroma

 **CAUTION.** Federal (USA) law restricts this device to sale by or on the order of a dentist.

**LOT NUMBER AND EXPIRATION DATE:**

Do not use after expiration date. The following numbers should be quoted in all correspondences:  
Order number, LOT number on the bottle, expiration date.

Symbol	Definition
	Manufacturer
	Medical Device
	Catalog Reference
	LOT Number
	Expiry Date


Symbol	Definition
	Temperature Limitation
	Humidity Limitation
	Shake Before Use
	Caution
	Follow Instructions For Use

Manufactured for:  
**Hu-Friedy Mfg. Co., LLC**  
3232 N. Rockwell Street  
Chicago, IL 60618 | USA  
1-800-Hu-Friedy | HuFriedyGroup.com

 **PRISMAN Inc.**  
1500 Broadway, Suite 1902 New York, NY 10036 | USA

Made in Germany  
HFL-902 / REV 3 - 2411  
Hu-Friedy® and HuFriedyGroup™ are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.  
All other product and company names referenced are trademarks of their respective owner.  
©2024 Hu-Friedy Mfg. Co., LLC. All rights reserved.

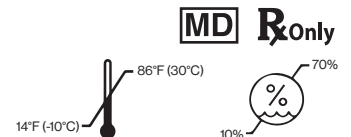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©2024 Hu-Friedy Mfg. Co., LLC. Tous droits réservés.