


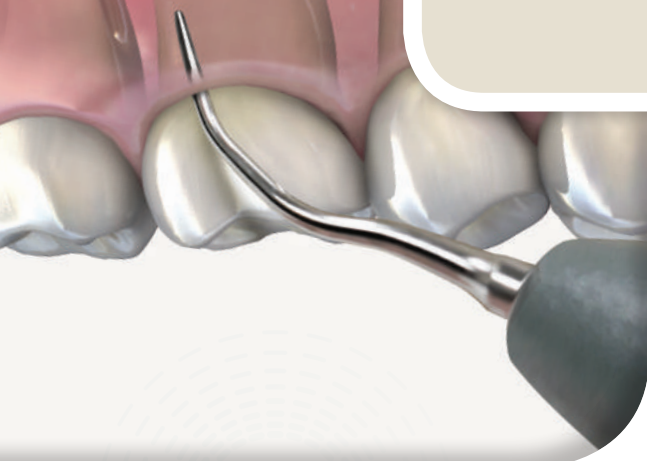

XT TRIPLE BEND INSERT

NEW FROM HU-FRIEDY



Joining Hu-Friedy's popular XT ultrasonic insert is the new **XT Triple Bend** – another Extra Thin insert to help you with your clinical power scaling needs.

XT Triple Bend Gives You:

- **Extra Thin** working end for superior accessibility in difficult areas
- **Beveled edges** for quick and effective calculus removal
- **Triple Bend** design for top notch adaptability to tooth surfaces

XT Triple Bend Clinical Uses:

- Ideal for patients with light to moderate deposits
- Great for interproximal areas, furcations and stain removal
- Following the removal of heavy deposits with your 1000 Standard Triple Bend, the XT Triple Bend can be used for finer power scaling on remaining light deposits and in tight intraoral spaces

WHICH INSERT IS RIGHT FOR YOU?

Proprietary blend of stainless steel alloy ensures the highest level of scaling efficiency


30 kHz Frequency

Through-tip water delivery for targeted water flow


Wide diameter handle

For **easy tip rotation** with a slight rolling of your fingertips

Swivel Direct Flow
XT Triple Bend
| UI30SDXTTB

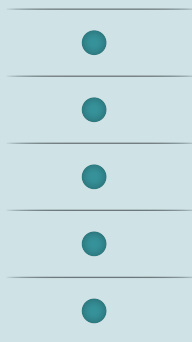

Streamline Direct Flow
XT Triple Bend
| UI30SFXTTB

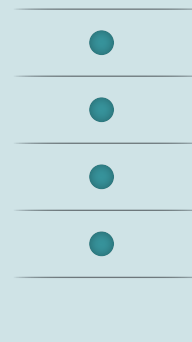

“Loved this instrument! It had **great water flow control**, was comfortable to grasp and **easily adapted** to difficult to reach areas.”


– Denise Wisham, RDH

EXPLORE HU-FRIEDY THIN INSERTS

Hu-Friedy offers several thin inserts, allowing improved access to subgingival areas, root anatomy, concavities, furcations and tight interproximal spaces.

XT INSERT

- Thin enough for hard-to-reach subgingival and furcation areas
- Ideal for tenacious calculus in narrow, deep pockets - a great site-specific addition to your set-ups!

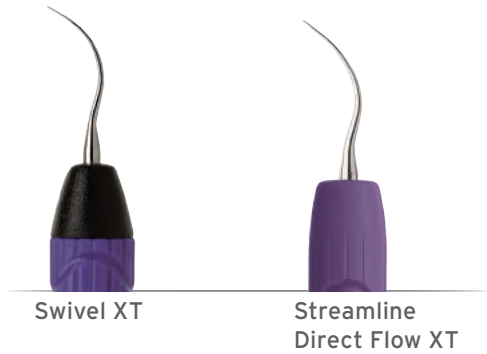

Swivel XT

Streamline Direct Flow XT

25 kHz Part Code: | **UI25SSXT**
30 kHz Part Code: | **UI30SSXT**

| **UI30SFXT**

#100 THIN INSERT

- Featuring a 40% thinner tip than Hu-Friedy's standard #10 insert
- Useful for generalized light to medium calculus and hard-to-reach areas



Swivel Direct Flow

Swivel

Streamline Direct Flow


| **UI25SD100**
| **UI30SD100**

| **UI25SS100**
| **UI30SS100**

| **UI30SF100**

AFTER FIVE® INSERTS – LEFT, STRAIGHT & RIGHT

- Reach deep into pockets with these probe-sized tips. Ideal for light to moderate deposits in periodontal pockets, furcations, developmental concavities, complex root anatomy and malpositioned teeth. After Five inserts are also efficient around fixed orthodontic appliances. With there many applications, After Five inserts can be an integral part of your procedural set-ups.

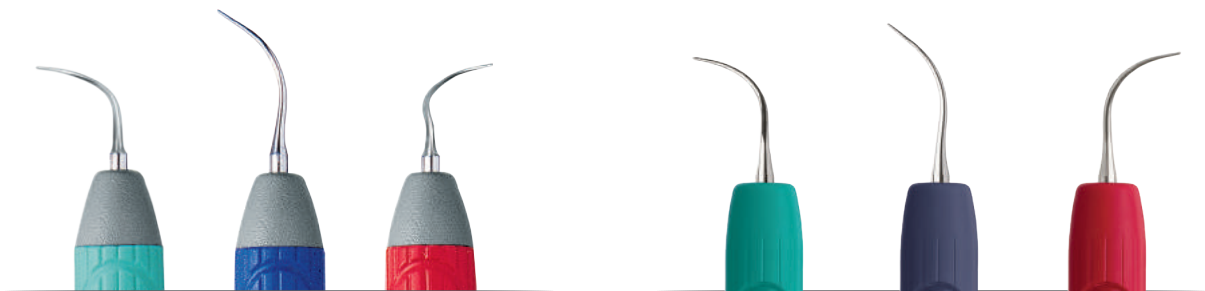

Left

Straight

Right

Left

Straight

Right

25 kHz Part Code: | **UI25SDL** | **UI25SDS** | **UI25SDR**
30 kHz Part Code: | **UI30SDL** | **UI30SDS** | **UI30SDR**

| **UI30SFL**

| **UI30SFS**

| **UI30SFR**

Swivel Direct Flow

Set of Three | **UI25SDSET** | **UI30SDSET**

Streamline Direct Flow

Set of Three | **UI30SFSET**

Visit us online at Hu-Friedy.com

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.
©2015 Hu-Friedy Mfg. Co., LLC. All rights reserved. *Cavitron is a registered trademark of Dentsply Inc. HF-511/0115

